PRO MASK

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 **Protective mask**

 **Atenschutzmaske**

FFP2

CE 1463

EN 149:2001+A1:2009

10 PCS

4 PLY

MADE IN EU

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Instruktion:

NOTE: Do not use the filtering half mask under the following conditions:

- in rooms with insufficient ventilation and small cubature, e.g. sewers, pits, tanks,
- in an atmosphere containing gaseous pollutants and vapors,
- facial hair may cause the mask to leak.

1. Wash your hands before putting on the mask and avoid touching the inside of the mask to reduce the risk of contamination.
2. Vary the inside and outside of the mask, and the top and bottom of the mask.
3. The mask should not be used during activities requiring considerable physical effort.
4. Pull the rubber bands in both directions, then pass them behind the ears and adjust until you feel comfortable.
5. Shape the nose clip to fit the shape of the nose to ensure a seal without excessive pressure. Adjust the position of the half mask so that the mask covers both the nose and chin.
6. Hold the mask as close to your face as possible. A simple test: breathe out after putting on the mask to keep air from escaping from the edge of the mask.
7. Before entering the workplace, check the tightness of the mask adhering to the face.
8. Try not to touch the mask with your hands. The mask can isolate biological contaminants on the surface. Touching the mask with your hands can transfer viruses, bacteria and other contaminants to your hands, which can cause infection.
9. During use, pay attention to the increase in breathing resistance until you notice that breathing has become noticeably difficult. If the breathing resistance is too high or the mask is mechanically damaged, replace it with a new one.
10. Be careful when removing the mask. Wash your hands thoroughly after doing this.
11. This is a single-use mask, so please throw it away after use.
12. Storage:
 - Filtering half-masks should be stored in rooms with a relative humidity of less than 80%, at a temperature of -4 °C to + 30 °C, in a closed, undamaged, individual manufacturer's packaging.
 - Half masks should be protected against sunlight and extreme temperatures.
 - Half-masks should be protected against aggressive chemicals, moisture and dirt.
 - The half masks are disposable and require no maintenance.
 - The shelf life of the half masks is 36 months.

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